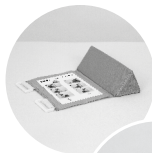
Distributed by

puras

031 996 85 85
puras.ch


Body Armor® Heel Reliever





EN Heel Protector
DE Fersen-Freilagerung
ES Soporte abierto para el talón
FR Dispositif de décharge du talon
IT Supporto aperto per tallone
NL Hielbeschermer
DA Hælsuspension
SV Hälavlastare
PL But odciążający piętę
SL Razbremenilnik pete
HR Ortoza za rasterećenje pete
PT Descanso do calcanhar

DARCO

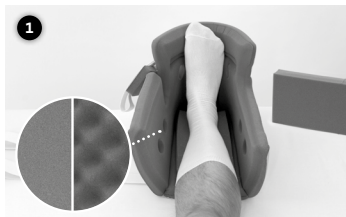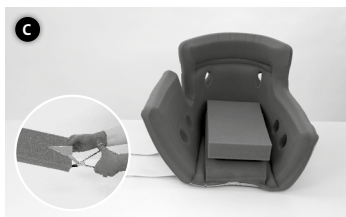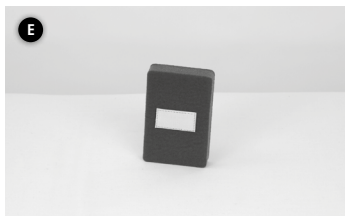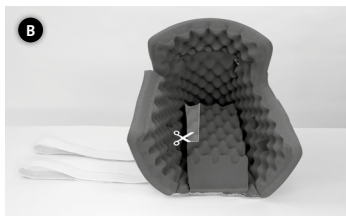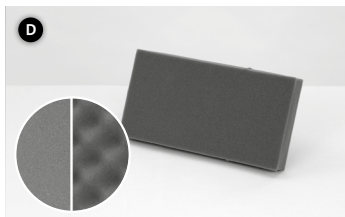Body Armor® Heel Reliever

Body Armor® Heel Reliever | Heel Protector

The Body Armor® Heel Reliever Heel Protector is for the prevention and healing of heel pressure sores as well as diabetic foot syndrome and decubitus.

1. Open the Body Armor® Heel Reliever Heel Protector. Remove the extra foam pad (photo D). Set the extra pad aside. Keep it available, you may need it later to customise the fit.
2. Place the patient's foot inside the support with the heel positioned above the heel suspension opening. The heel should hang over the fixed elevation pad (photos 1 and 2).
3. Pull the hook and loop straps across the shin towards the loops. Guide the straps through the loops. Secure the straps with the hook and loop fasteners (photo 3). For particularly narrow calves, allow the side panel to overlap.
4. Check the fit. There should be sufficient space for your fingers between the heel opening and the bed (photo 4). Do not fasten the Heel Protector too loosely or too tightly around the leg. The hook and loop straps should not be in contact with the skin. The forefoot pad (photo E) can be used optionally for protection between the hook and loop strap and foot (photo 5).
5. Instructions for use: Depending on patient / indication either use a Heel Protector with smooth foam or convoluted foam (photo 1).

Customising the fit:

- > **Foot drop:** Secure the extra pad in a vertical position behind the sole of the foot (photo A).
- > **Malleolar decubitus (ankle ulcer):** Cut away the nubs surrounding the ankle on the inside of the support. Cut a portion of the fixed foam pad if necessary (photo B).
- > **Hip rotation:** Secure the positioning wedge (photo F) to the Heel Protector, according to use.
- > **Bariatric legs:** If the heel comes into contact with the bed sheet, add the extra pad on top of the fixed pad for increased elevation.
- > **Heel cord:** If redness occurs in this area, cut a "V" shape out of the fixed pad. If using both pads, customize the upper foam pad (photo C).

The pressure on the heel is redistributed to the heel cord and gastrosoleus complex. The Body Armor® Heel Reliever Heel Protector should be removed and the leg and foot inspected for skin changes, every 6 – 8 hours. Should redness (erythema) occur, a V-shaped wedge should be removed from the fixed pad in the area of the heel cord. Oedematous patients should use the smooth Body Armor® Heel Reliever version to avoid skin irritation.

Indications

- > For the prevention and healing of heel pressure ulcers as well as diabetic foot syndrome and decubitus
 - > Braden scale scores of fourteen or less
 - > Diabetes with neurological sensory deficit
 - > No palpable pedal pulses
 - > Paraplegia
 - > Prolonged immobility or coma
- Contraindications:**
- > Wounds on the calf
 - > Spasticity
 - > Contraction of the ankle

Can be used universally on the right and left side (25 – 54 cm calf circumference)

Colour: Grey-Blue

Material: **Heel suspension:**

Outer cover: Polyester

Foam: Latex-free polyurethane foam

Hook and loop straps: Nylon

Forefoot pad: Polyester, latex-free polyurethane foam

Extra pad: Latex-free polyurethane foam

Positioning wedge:

Cover: Polyester

Wedge: Ethylene vinyl acetate copolymer

Laundry bag: Polyester

Maintenance: The product does not require maintenance

Design features: The Body Armor® Heel Reliever Heel Protector is available in 2 versions: convoluted or smooth. Both product variants are laminated with a smooth fabric cover to allow the leg to “glide” without changing the position of the exposed area. The correct fixation of the product on the patient is achieved with three straps that hold the forefoot and the lower leg in an orthograde position. If the leg is rotated excessively over the hip joint, the desired position of the leg can be achieved with the positioning wedge. An extra pad for the calf area and the plantar foot area included in the scope of delivery enables an individual, pressure redistributing and safe application of the product. The Body Armor® Heel Reliever Heel Protector can be used on the left or right foot.

Cleaning instructions

Hand wash:

- > Use mild detergent and warm water. Allow to dry at room temperature.

Machine wash:

- > Close hook and loop straps and wash in the laundry bag (photo G) or pillowcase provided (60°C). Allow to dry at room temperature.
- > Disinfection after washing: With an antiseptic spray

Water, standard cleaning agents and disinfectants can be used for cleaning. The hook and loop strips should always remain closed during cleaning.

Caution

Please always follow these instructions for use. In case of adverse effects (e.g. skin swelling, pressure points, disorders of blood circulation etc.) please tell your doctor. Function and optimum protection are no longer guaranteed if the product is handled incorrectly. This aid assists recovery and should be used exclusively under the instruction of and in consultation with your doctor or a healthcare professional. DARCO (Europe) GmbH does not assume any responsibility. Do not wear the product with injured, irritated, damaged or intact skin. You should use bandages or socks, for example. This aid is intended for single-patient use only. The product is only intended for support. Patients should not walk when wearing the product.

Disposal:

- > Discard after use
- > Please dispose of the packaging in an environmentally friendly manner. The product can be disposed of with household waste.

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