**en**  
**INDICATIONS:** For use on adhesives, dressings and appliances on skin.  
**DESCRIPTION OF DEVICE & CONTENTS:** ESENTA™ medical adhesive remover wipes sting free

**INTENDED USE:** Removal of adhesive residues from skin

**INSTRUCTION FOR USE:**

1. Tear sachet to open and unfold wipe
2. Rub the wipe over adhesive residues to remove

**INFORMATION:** Dries in a few seconds. Create a clean surface without affecting the adhesion of the next appliance or dressing.

**CAUTION:**

This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations.

If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

- Avoid direct use on wound beds.

- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).

- If in doubt, patch test a small area prior to use.

- Should skin irritation occur, discontinue use and consult a healthcare professional.
- For external use only.

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ARTWORK SPECIFICATION	
Artcode	SI2383
Artwork Description	INS ESENTA ADHESIVE REMOVER WIPE 30PKINT
ICC	423281, 423391
SAP	1729194, 1733699
KOM/REF	N/A
IS Code	N/A
Component Type	Insert
LR#	LR-004248
Existing Art Code being Revised	N/A
REQUIREMENT	SPECIFICATION
Bucket	INT
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 396mm x 210mm, Folded: 66 mm x 105mm
Color(s) / Colour(s)	Black
Specification Reference(s)	TP/TF21-052
Bar Code: UPC (A) or GTIN*	N/A
Bar Code: PZN – Code 39*	N/A
Bar Code: JAN-13*	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
<b>HISTORY</b>	
28-Jan-2021 / SI2383: New insert for Esenta Adhesive Remover Wipe.	

WI-0010, Ver. 6.0

<b>Workflow Name</b>	<b>Workflow Start Date</b>	<b>User</b>	<b>Intention</b>	<b>Date</b>	<b>Justif.</b>
Workflow - LBL Approval with Proof Review Mar 29, 2021	29-Mar-2021 13:00:46 (GMT)	Milada Palkociova	Final Quality Approval	30-Mar-2021 16:34:55 (GMT)	Final Artwork Check
Workflow - LBL Approval with Proof Review Mar 29, 2021	29-Mar-2021 13:00:46 (GMT)	Milada Palkociova	Quality Approval	12-Apr-2021 13:10:27 (GMT)	Plant Review-1
Workflow - LBL Approval with Proof Review Mar 29, 2021	29-Mar-2021 13:00:46 (GMT)	Milada Palkociova	Quality Approval	12-Apr-2021 13:09:49 (GMT)	Proof Review

Workflow - LBL Approval with Proof Review Mar 29, 2021	29- Mar-2021 13:00:46 (GMT)	Iveta Bajužíková	Quality Approval	11- May-2021 23:14:12 (GMT) Implementation	Plant Review and Implementation
Workflow - LBL Approval with Proof Review Mar 29, 2021	29- Mar-2021 13:00:46 (GMT)	Milada Palkociova	Final Quality Approval	08- Apr-2021 12:56:29 (GMT)	Final Artwork Check