(en)

Indications: For use on skin around stomas and skin around wounds.
DESCRIPTION OF DEVICE & CONTENTS: ESENTA™ skin barrier spray sting free (50mL)
INTENDED USE: Protection of skin from the damage associated with body waste, enzymes and adhesives.
INSTRUCTION FOR USE:

- Clean and dry the skin.
- Hold approximately 10cm from the area to be treated.
- Apply a uniform coating in a sweeping motion.
- Ensure the area is completely dry before applying adhesive products such as appliances or dressings.
- Replace at every dressing or appliance change to maintain complete protection.

INFORMATION: Removal is not normally necessary. However, ESENTA™ skin barrier can be removed easily using ESENTA™ medical adhesive remover. Sprays at any angle. Soothes reddened and sore skin. Reduces itching. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

CAUTION:

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- Avoid inhalation.
- For external use only.

WARNINGS: Use only as directed. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Do not pierce or burn even after use. Protect from sunlight. Do not expose to temperatures exceeding 50°C/122°F. Keep out of reach of children. EXTREMELY FLAMMABLE

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ARTWORK SPECIFICATION

Artcode	SI2388
Artwork Description	INS ESENTA SKIN BARRIER SPRAY (50ML)INT
ICC	423288
SAP	1729205
KOM/REF	N/A
IS Code	N/A
Component Type	Insert
LR#	LR-004247
Existing Art Code being Revised	N/A
REQUIREMENT	SPECIFICATION
Bucket	INT
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 396 mm x 200mm, Folded: 33mm x 200mm
Color(s) / Colour(s)	Black
Specification Reference(s)	TP/TF21-053
Bar Code: UPC (A) or GTIN*	N/A
Bar Code: PZN – Code 39*	N/A
Bar Code: JAN-13*	N/A

* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.

HISTORY

28-Jan-2021 / SI2388: New insert for Esenta Skin Barrier spray.

Workflow Name	Workflow Start Date	User	Intention	Date	Justif.
Workflow - LBL Approval with Proof Review Apr 7, 2021	07-Apr-2021 05:00:54 (GMT)	Iveta Bajuziková	Quality Approval	11-May-2021 23:11:03 (GMT)	Plant Review and Implementation
Workflow - LBL Approval with Proof Review Apr 7, 2021	07-Apr-2021 05:00:54 (GMT)	Milada Palkociova	Quality Approval	09-Apr-2021 09:26:21 (GMT)	Plant Review-1
Workflow - LBL Approval with Proof Review Apr 7, 2021	07-Apr-2021 05:00:54 (GMT)	Milada Palkociova	Quality Approval	09-Apr-2021 09:25:27 (GMT)	Proof Review

Workflow - LBL Approval with Proof Review Apr 7, 2021	07- Apr-2021 05:00:54 (GMT)	Marian Vinc	Final Quality Approval	07- Apr-2021 05:12:32 (GMT)	Final Artwork Check
Workflow - LBL Approval with Proof Review Mar 31, 2021	31- Mar-2021 10:02:24 (GMT)	Marian Vinc	Final Quality Approval	01- Apr-2021 13:04:17 (GMT)	Final Artwork Check