en

**Indications:** For use on skin around stomas and skin around wounds.

**DESCRIPTION OF DEVICE & CONTENTS:** ESENTA™ skin barrier wipes sting free

**INTENDED USE:** Protection of skin from the damage associated with body waste, enzymes and adhesives.

**INSTRUCTION FOR USE:**

- Clean and thoroughly dry the skin
  - Tear sachet to open and unfold wipe
  - Gently wipe the area to be protected and wait until dry
- INFORMATION:** Dries in a few seconds. Create a clean surface without affecting the adhesion of the next appliance or dressing.Soothes reddened and sore skin. Reduces itching.

**CAUTION:** This device is for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. After use, this product may be a potential biohazard. Handle and dispose of in accordance with accepted medical practice and applicable local laws and regulations. If during the use of this device or as a result of its use a serious incident has occurred, please report it to the manufacturer and to your national authority.

- Avoid direct use on wound beds.
- Avoid use on delicate or sensitive areas of skin (eyes, mouth, ears, nose, vagina).
- If in doubt, patch test a small area prior to use.
- Should skin irritation occur, discontinue use and consult a healthcare professional.
- For external use only.

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**sv**

**tr**

ARTWORK SPECIFICATION	
Artcode	SI2384
Artwork Description	INS ESENTA SKIN BARRIER WIPE 1X30PK INT
ICC	423282, 423392
SAP	1729195, 1733700
KOM/REF	N/A
IS Code	N/A
Component Type	Insert
LR#	LR-004248
Existing Art Code being Revised	N/A
<b>REQUIREMENT</b>	<b>SPECIFICATION</b>
Bucket	INT
Local Code(s)	N/A
Size Code / Dieline	N/A
Dimensions	Flat: 396mm x 210mm, Folded: 66 mm x 105mm
Color(s) / Colour(s)	Black
Specification Reference(s)	TF21-051
Bar Code: UPC (A) or GTIN*	N/A
Bar Code: PZN – Code 39*	N/A
Bar Code: JAN-13*	N/A
* NOTE: Bar Code: See artwork for positioning, magnification and truncation. Scanability verified to ANSI grade "C" or better.	
<b>HISTORY</b>	
28-Jan-2021 / SI2384: New insert for Esenta Skin Barrier Wipe.	

<b>Workflow Name</b>	<b>Workflow Start Date</b>	<b>User</b>	<b>Intention</b>	<b>Date</b>	<b>Justif.</b>
Workflow - LBL Approval with Proof Review Mar 30, 2021	30-Mar-2021 10:12:42 (GMT)	Milada Palkociova	Final Quality Approval	31-Mar-2021 11:49:26 (GMT)	Final Artwork Check
Workflow - LBL Approval with Proof Review Mar 30, 2021	30-Mar-2021 10:12:42 (GMT)	Milada Palkociova	Quality Approval	12-Apr-2021 13:13:38 (GMT)	Plant Review-1
Workflow - LBL Approval with Proof Review Mar 30, 2021	30-Mar-2021 10:12:42 (GMT)	Milada Palkociova	Quality Approval	12-Apr-2021 13:12:17 (GMT)	Proof Review

Workflow - LBL Approval with Proof Review Mar 30, 2021	30- Mar-2021 10:12:42 (GMT)	Iveta Bajuziková	Quality Approval	11- May-2021 23:15:09 (GMT)	Plant Review and Implementation
Workflow - LBL Approval with Proof Review Mar 30, 2021	30- Mar-2021 10:12:42 (GMT)	Milada Palkociova	Final Quality Approval	08- Apr-2021 12:22:40 (GMT)	Final Artwork Check
Workflow - LBL Approval with Proof Review Mar 30, 2021	30- Mar-2021 10:12:42 (GMT)	Beata Lazurova	Final Quality Approval	07- Apr-2021 07:11:36 (GMT)	Final Artwork Check