





Etac Swift/Etac Smart/ Etac Edge/Etac Easy Manual

etac®

Distributed by
puras

031 996 85 85
puras.ch

24-04-2012 78390H

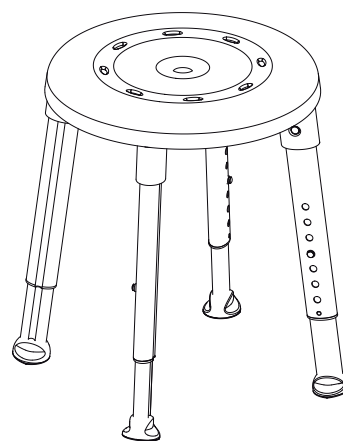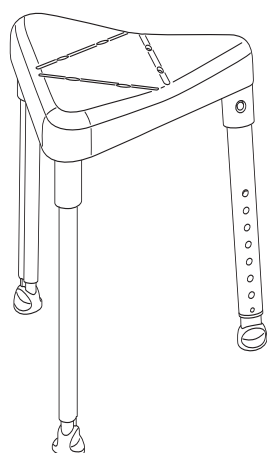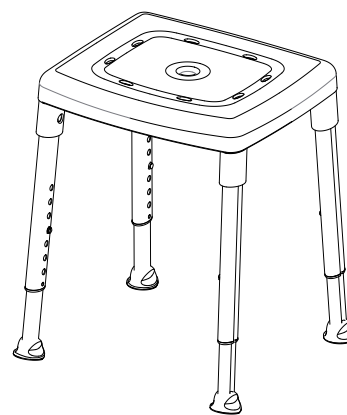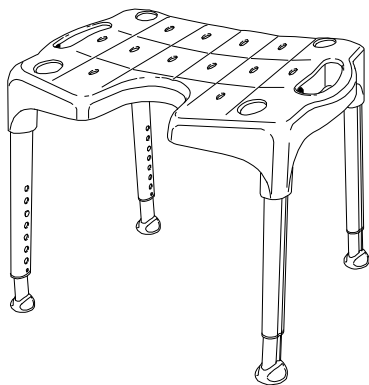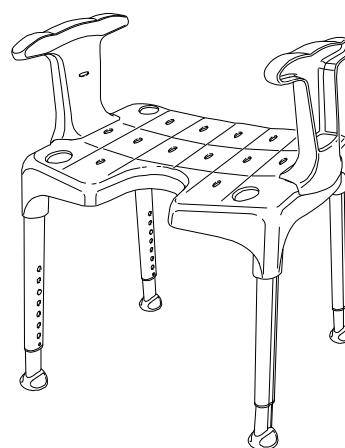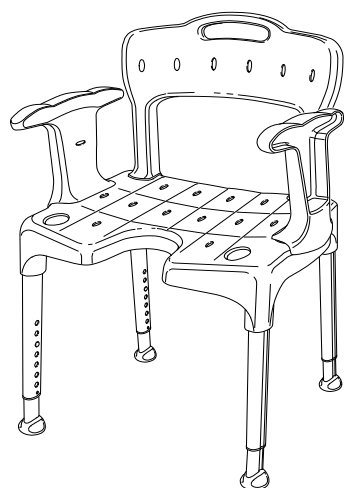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EN

Intended use

Facilitates sitting when carrying out intimate hygiene.

Description

Our stools/chairs are designed for use in showers or in bathrooms. The products have a stable design with built-in flexibility which adapt to uneven floors. All products are fitted with height-adjustable legs that are simple to attach. The leg ferrules offer good grip and the anti-slip pattern of the seat offer the important safety and stability properties that make the user feel safe. Etac Swift can be anything from a simple shower stool to a complete shower chair, depending on how you choose to equip it.

Mounting

The products are easy to assemble. No tools are needed.

Accessories

Etac Swift

Armrest/pair: Swift can be easily fitted with armrests for better support when standing up.

Back support: Can be easily fitted to offer users better stability.

Seat pad: A seat pad that provides both softness, warmth and friction.

Backrest pad: A back pad that provides softness, warmth and friction.

Soap dish: A small shelf to keep your shampoo, soap or shower gel, within convenient reach. Shower head holder is included.

Etac Edge, Etac Smart, Etac Easy

Seat pad: A seat pad that provides both softness, warmth and friction.

Safety

Warning: A defective product must not be used.

Warning: A height difference of more than one hole is not allowed for chair legs.

Warning: When moving sideways, do not lean on just one armrest.

Max user weight: Swift/Edge 130 kg, Smart/Easy 150 kg.

Cleaning: Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

Decontamination: Can be decontaminated at max. 85° C/185°F for 3 minutes if required.

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Etac Sverige AB
Etac AB (export)
Etac A/S