





# Etac Swift/Etac Smart/ Etac Edge/Etac Easy Manual

**etac**<sup>®</sup>

Distributed by  
**puras**

031 996 85 85  
puras.ch

24-04-2012 78390H

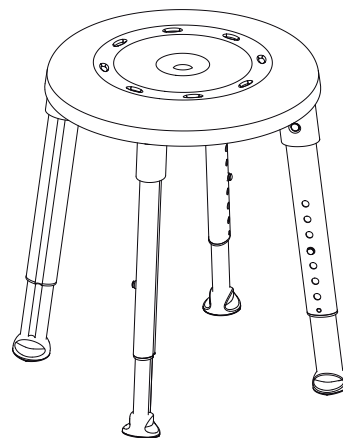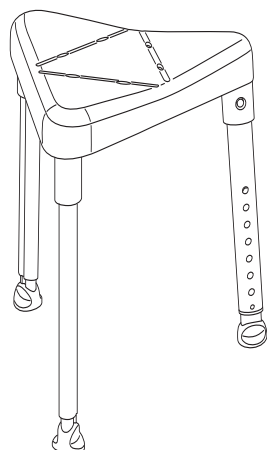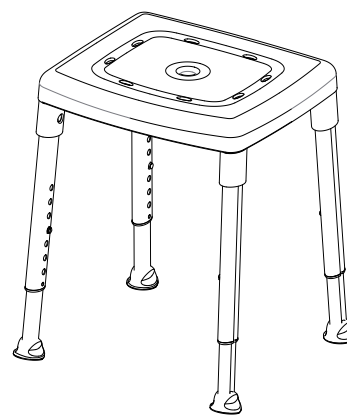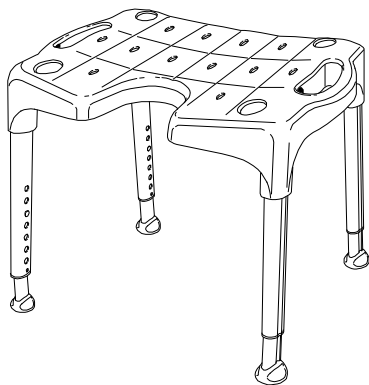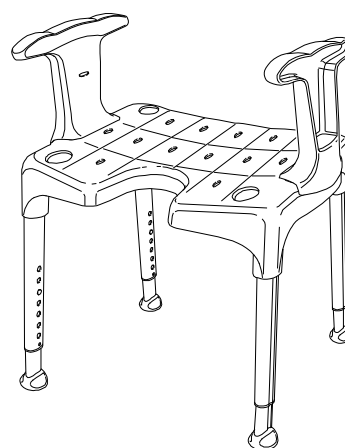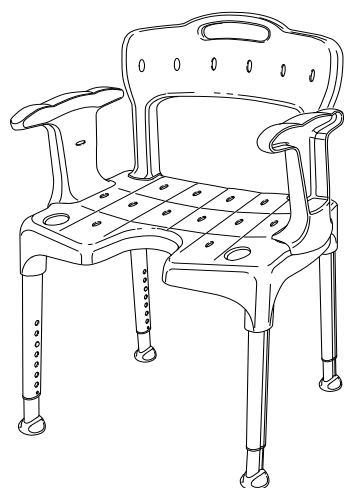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EN

### Intended use

Facilitates sitting when carrying out intimate hygiene.

### Description

Our stools/chairs are designed for use in showers or in bathrooms. The products have a stable design with built-in flexibility which adapt to uneven floors. All products are fitted with height-adjustable legs that are simple to attach. The leg ferrules offer good grip and the anti-slip pattern of the seat offer the important safety and stability properties that make the user feel safe. Etac Swift can be anything from a simple shower stool to a complete shower chair, depending on how you choose to equip it.

### Mounting

The products are easy to assemble. No tools are needed.

### Accessories

#### Etac Swift

**Armrest/pair:** Swift can be easily fitted with armrests for better support when standing up.

**Back support:** Can be easily fitted to offer users better stability.

**Seat pad:** A seat pad that provides both softness, warmth and friction.

**Backrest pad:** A back pad that provides softness, warmth and friction.

**Soap dish:** A small shelf to keep your shampoo, soap or shower gel, within convenient reach. Shower head holder is included.

#### Etac Edge, Etac Smart, Etac Easy

**Seat pad:** A seat pad that provides both softness, warmth and friction.

### Safety

**Warning:** A defective product must not be used.

**Warning:** A height difference of more than one hole is not allowed for chair legs.

**Warning:** When moving sideways, do not lean on just one armrest.

**Max user weight:** Swift/Edge 130 kg, Smart/Easy 150 kg.

**Cleaning:** Clean the product using a solvent-free cleaning agent with a pH value of 5-9 or with a 70% disinfectant solution.

**Decontamination:** Can be decontaminated at max. 85° C/185°F for 3 minutes if required.

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Etac Sverige AB
Etac AB (export)
Etac A/S