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Norm: DIN EN ISO 999 (08/98)

Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Do not lean out of the raised seat to avoid falling out of the device.
- ♥ Do not seat or step on the lid.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!).
- ♥ Don't use outside but only inside and on standard toilet bowls.
- ♥ Use only as a raised toilet seat and in proper way : don't knee or step on the product, don't seat on the lid.
- ♥ ALWAYS observe the weight limit on the labelling of your device.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ After assembling the product, make sure that it is securely locked and fixed before using.
- ♥ In case you didn't use your raised seat for a long time, let it checked by the technical staff, before using.
- ♥ Always check stability before using.

2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs.

Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Do not use the product if you see any risk.

- 1-Lift the toilet seat.
- 2-Push the raiser downwards on the toilet bowl until it fits.
- 3-Check that the «lips» of the raiser are covering the toilet bowl edge.
- 4-Check stability before use.
- 5-Remove simply the raiser to clean and restart from point 1 to install.

Rental: this product can be rented under the condition that it was correctly cleaned. This user's manual belongs to that product and must be delivered to each new user.

3. Cleaning

The toilet seat raiser has to be cleaned daily with a non-abrasive cleaner. It can be cleaned with products containing no chlorine, such as Surfanios (Anios laboratories), or with soapy water. Do not use any corrosive or abrasive detergents, or pressure cleaners. Sterilisation is possible at 90°C (194°F) and 80 % humidity for 5 minutes.

4. Materials

The seat is made of one piece, the skin is made of plastic and the inside made out of foam.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect. This product is biocompatible.

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