EN Kocher forceps

INTENDED PURPOSE, INDICATIONS AND CLINICAL BENEFIT

Kocher forceps is intended to be used for:

- Grasping skin and soft tissues in woundcare procedures.
- Holding swabs for absorption of blood and fluids during surgery or woundcare procedures.
- Holding swabs for skin disinfection procedures.

WARNINGS AND PRECAUTIONS

- Kocher forceps is to be used exclusively by healthcare professionals.
- Kocher forceps is not intended to be used for hemostasis or to clamp vessels.
- Do not re-use. Kocher forceps is intended to be used one single time to prevent risks from contamination, including possible infections, and/or functional damage.
- Visually inspect the device. Do not use the device if you detect any sign of deterioration or defect that may compromise the performance.
- Visually inspect the sterile barrier system prior to use. Do not use the device if you detect any breach in packaging integrity.
- Dispose according to local laws rules, regulations and/or infection prevention standards for healthcare waste, to minimize potential risk of contamination, infection or pollution.

The occurrence of serious incidents is not expected with the use of Kocher forceps if used correctly, but according to Regulation (EU) 2017/745 we must inform that serious incidents related with the use of the device, that are life-threatening or compromise health, including illness or permanent impairment of a body structure, must be notified to Bastos Viegas and to the Competent Authority of your Country.

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