Distributed by

**puras**

031 996 85 85  
puras.ch

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*User Manual*

### **CUSCINO IN LATTICE »ROTONDO«**

COUSSIN EN LATEX »ROND«

LATEX CUSHION »ROUND«

*Art.N./ N°art./ Item No.: **41 000 - 41 008***

---

### **CUSCINO IN LATTICE »OVALE«**

COUSSIN EN LATEX »OVALE«

LATEX CUSHION »OVAL«

*Art.N./ N°art./ Item No.: **41 050 - 41 058***

**sundo**  
HOMECARE

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### **EN**

#### **Intended use:**

This latex pillow is a medical device and is designed for domestic use to provide short-term pressure relief in the event of seating problems.

#### **Indication:**

Aid for relieving pressure and improving physiological seating positions in acute situations. The latex cushion can noticeably improve seating problems in the event of episiotomies and (postoperative or conservative) prostate, coccyx, anal/hemorrhoidal or bladder indications. The patient must be able to move in order to relieve the pressure, if necessary.

#### **Contraindications:**

Do not use the latex cushion over long periods of time as this may lead to an oedema in the area over the cushion's hole.

#### **How to use:**

Latex cushions should be used to provide patients with maximum relief in the sensitive area of the body.

The hole in the centre of the cushion provides a hollow space under the affected area of the body.

The cushion should only be used with the cover included or an alternative, breathable cover. When storing, make sure there are no wrinkles in the cover to avoid pressure points.

#### **Before use/Maintenance:**

Firstly, please read the instructions for use in their entirety and store them with the packaging until the end of the product's service life. Check that the product is undamaged before the first use and each time it is used. If the cushion shows signs of damage or wear and tear, stop using it and contact your retailer.

The product must only come into contact with intact, unin-

jured skin and must not be used for more than one patient. Use going beyond what is mentioned above or any other use is not in accordance with the intended use. Sundo GmbH is not liable for any resulting damage; the risk is borne exclusively by the user. The manufacturer or competent authority must be notified immediately of all serious incidents relating to this product.

#### **Cleaning and Disinfection:**

The cover of the latex cushion must be washed before the first use

(up to max. 40 °C in the washing machine).

- ▶ The latex cushion itself can be washed by hand at temperatures of up to 30° C if necessary. Allow to air dry; do not spin dry, tumble dry or dry on a heater. If necessary, the cushion can be disinfected with a disinfecting agent suitable for commercial latex. The cover can be treated with suitable disinfecting agents or a hygienic rinsing agent. Important: Pay attention to the manufacturer's instructions! Caution, not suitable for use across multiple patients!

#### **⚠ Safety instructions:**

- ▶ This latex cushion contains natural rubber latex, which may cause allergic reactions in sensitive individuals under certain circumstances! In the event of an allergic reaction, stop using the cushion immediately and use a viscoso foam pad or similar instead!
- ▶ Latex is sensitive to light, so always use and/or store the cushion with the cover provided!
- ▶ Heavy use such as pulling, tearing and contact with sharp or pointed objects should be avoided to prevent damage to the cushion.
- ▶ Caution, do not smoke when using or storing, keep away from sources of heat, risk of fire!

#### **Warranty:**

In accordance with statutory warranty rights. Excluded from the warranty: Damage caused by improper use, normal wear and tear, and defects which do not significantly affect the value or usability of the product. Technical modifications and attachments, as well as improperly performed maintenance void the warranty and product liability in general. We reserve the right to make modifications aimed at improving technical features.