## Oxygen Tubing Swivel Connectors

**REF** 1221-0, 1222-0

### EN Oxygen Tubing Swivel Connectors

**DEVICE DESCRIPTION:** *The Swivel connector* has ribbed body design and male conical tip. This device is non-sterile, disposable, and for single-patient use.

**INTENDED PURPOSE:** *Swivel connector*; Intended to join two lengths of oxygen supply tubing.

**INDICATIONS FOR USE:** *Swivel connector*; Join two oxygen supply tubings.

**ENVIRONMENT:** Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities
The swivel connectors with metal clips, 1221-0 & 1222-0, are MRI Unsafe.

**PATIENT TARGET GROUP:** Infant, pediatric, adult. Spontaneously breathing individuals with a variety of breathing conditions requiring supplemental oxygen.

**EXPECTED CLINICAL BENEFITS:**

(Indirect)

The tubing connectors are part of the oxygen setup to deliver supplemental oxygen from gas outlet to patient interface.

- The Swivel connector*; has a 360 degree rotation that helps to reduce tubing torque and twisting preventing interruption of flow.

**CONTRAINDICATIONS:** None Known.

**WARNINGS:**

- If oxygen is in use, do not use near flame or heat source.
- Device contains components that may present choking hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.
- The swivel connectors with metal clips, 1221-0 & 1222-0, are MRI Unsafe.

**CAUTIONS:** None Known.

**RESIDUAL RISKS:** Refer to Warnings and Cautions.

**NECESSARY USER QUALIFICATIONS:** This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

**INSTRUCTIONS:** Follow instructions to avoid risk of infection and contamination. Check for leaks at connections and for proper placement prior to use. This product is disposable and not intended to be cleaned, reused, or for prolonged use (≥3 months). Discard and replace if the tubing becomes soiled, discolored, or stiff, or at 3 months. Discard and replace the device if patient has had a respiratory infection. Use water trap if excess condensation is present.

**INSTRUCTIONS FOR USE IN COMBINATION WITH ACCESSORIES / OR OTHER DEVICES:**

*Oxygen Tubing Swivel Connectors* are for use within an oxygen delivery system and connects tubing to tubing.

- Connect one end of the tubing to the oxygen source.
- Insert, push, and twist the tapered end of the tubing connector into oxygen tubing end connector.
- Repeat to attach the other side of the connector to the nasal cannula or oxygen mask tubing.
- Gently tug on connection to check for a secure fit.
- To remove, hold the center of the tubing connector, twist and pull on the tubing end connector.

**SAFE DISPOSAL:** Dispose of device in accordance with local, state or national regulations.

**INCIDENT REPORTING:** Contact your country’s Competent Authority and Manufacturer to report any serious incident.

**PERFORMANCE CHARACTERISTICS:**

- Fits Oxygen Tubing with standard 5mm to 7mm end connectors.
- Flow rate: 0-15 LPM.

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