Oxygen Tubing Connectors

EN Oxygen Tubing Connectors

DEVICE DESCRIPTION:

- The Nipple Nut connector*** is a tapered and ribbed, one piece, nipple nut assembly for threaded DISS gas fittings.
- The Swivel/Static connector*** has ribbed body design and male conical tip.
- The Y-connector*** has an open ribbed male connector, and the other two ends are bonded to green, 3-channel tubing.

These devices are non-sterile, disposable, and for single-patient use.

INTENDED PURPOSE:

- Nipple Nut connector:*** Intended to connect oxygen tubing to a gas source.
- Swivel/Static connector:*** Intended to join two lengths of oxygen supply tubing.
- Y-connector:*** Intended to connect two oxygen sources to one supply tubing.

INDICATIONS FOR USE:

- Nipple Nut connector:*** Connect oxygen supply tubing to a DISS 1240 fitting.
- Swivel/Static connector:*** Join two oxygen supply tubings.
- Y-connector:*** Join two oxygen sources to deliver higher flow rates to patient interface.

ENVIRONMENT: Hospitals, sub-acute, medical clinics, pre-hospital, home, surgical centers, skilled nursing facilities. Static connector, Y-connector, In-Line Water Trap, and Swivel connector without metal clips are MRI safe.

PATIENT TARGET GROUP: Infant, pediatric, adult. Spontaneously breathing individuals with a variety of breathing conditions requiring supplemental oxygen.

EXPECTED CLINICAL BENEFITS:

(Indirect)

The tubing connectors are part of the oxygen setup to deliver supplemental oxygen from gas outlet to patient interface.

- The Swivel*** connector has a 360 degree rotation that helps to reduce tubing torque and twisting preventing interruption of flow.

- The connectors join oxygen tubing to increase patient’s mobility and improve health related quality of life.

- The Y-Connector*** joins two oxygen sources allowing for higher flow rates to relieve hypoxemia.

CONTRAINDICATIONS: None Known.

WARNINGS:

- If oxygen is in use, do not use near flame or heat source.
- Device contains components that may present choking hazard.
- To reduce the risk of misconnections and patient injury, always trace tubing from gas source to the medical device before connecting.

CAUTIONS: None Known.

RESIDUAL RISKS: Refer to Warnings and Cautions.

NECESSARY USER QUALIFICATIONS: This device is restricted to sale by or on the order of a physician. There are no additional requirements for special facilities, special training, or particular qualifications for the use of this device.

INSTRUCTIONS: Follow instructions to avoid risk of infection and contamination. Check for leaks at connections and for proper placement prior to use. This product is disposable and not intended to be cleaned, reused, or for prolonged use (≥3 months). Discard and replace if the tubing becomes soiled, discolored, or stiff, or at 3 months. Discard and replace the device if patient has had a respiratory infection. Use water trap if excess condensation is present.

INSTRUCTIONS FOR USE IN COMBINATION WITH ACCESSORIES / OR OTHER DEVICES:

1) Nipple Nut Connector is for use within an oxygen delivery system and connects tubing to a DISS 1240 fitting.

1. Attach the nipple adaptor to oxygen gas source.

2. Turn wing nut until tight.

3. Attach oxygen supply tubing to barbed nipple end.

2) Oxygen Tubing Connectors (Swivel/Static) are for use within an oxygen delivery system and connects tubing to tubing.

1. Connect one end of the tubing to the oxygen source.

2. Insert, push, and twist the tapered end of the tubing connector into oxygen tubing end connector.

3. Repeat to attach the other side of the connector to the nasal cannula or oxygen mask tubing.

4. Gently tug on connection to check for a secure fit.

5. To remove, hold the center of the tubing connector, twist and pull on the tubing end connector.

3) Y-Connector is used to connect two oxygen sources to one supply tubing.

1. Connect the tubing on the Y-connector to the two oxygen sources.

2. Insert the barb end of the Y-connector into the end of nasal cannula or oxygen mask tubing.

3. Gently pull on the connection to ensure secure fit.

4. Check connections for leaks.

5. To remove, hold the center of the Y-connector while pulling on the nasal cannula or mask tubing.

6. Disconnect the tubing from oxygen sources.

SAFE DISPOSAL: Dispose of device in accordance with local, state or national regulations.

INCIDENT REPORTING: Contact your country’s Competent Authority and Manufacturer to report any serious incident.

PERFORMANCE CHARACTERISTICS:

• Fits Oxygen Tubing with standard 5mm to 7mm end connectors.

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MT Promedt Consulting GmbH