Salvafix®

Memory^{PRO}

Standard

Slim

Transit

- ES Español
- EN English
- DE Deutsch
- FR Français
- IT Italiano
- PT Português

- CA Català
- NL Nederlands
- PL Polski
- LT Lietuvos
- ZH 中文
- JA 日本語

WARNING

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ENGLISH

INSTRUCTIONS FOR USE:

Please refer to the illustrations in this leaflet.

SAFETY RECOMMENDATIONS:

1. Use the size of product most suitable for the Patient (please see measurements table).
2. Please fully check the **ENTIRE** system **BEFORE USE**.
3. Firmly fix all Salvafix straps to the frame of the bed. In articulated beds always attach them to the moving frames.
4. **ALWAYS** use the two side bands of the Salvafix belt.
5. Use Salvafix waist belt with **PERINEAL (A1)** or **FEMORAL** bands (**A2**) in order to prevent patient from sliding up and down.
6. The **TWO** side guardrails of the bed must be always kept in the **UPPER** position and **BLOCKED**. **B**
7. Constantly monitor all Patients that are being restrained by means of Salvafix mechanisms. **C**
8. Follow all the instructions contained in the Restraint Protocol of your place of work and according the laws of your country.
9. **DO NOT USE** Salvafix if you have not been fully informed and trained in its use.
10. A quick and safe cutter should always be on hand for emergency cases. **D**
11. The magnetic keys **MUST NEVER** be taken nearer than 15cm to Patients with a pace-maker.
12. Do not submerge and avoid liquids into the lock buttons **SVF3500 - SVF3510**.
13. Salvafix products must always be used with original Salvafix locking systems included in the package and/or **SVF3500, SVF3505, SVF3510 and SVF3520**.

- **DO NOT USE** Salvafix products if you do not comply with **ALL** these points.
- Improper use of Salvafix restraint systems can cause serious injury or even death by strangulation or abdominal asphyxia. **E**
- Manufacturer would not be liable when any of this recommendations are not met.

MAINTENANCE INSTRUCTIONS:

- Salvafix products are not fireproof.
- Before washing / drying Salvafix restraint products, the Salvafix locking buttons and anchoring pins should be removed.
- Salvafix products are made of 100% polyester and are resistant when washed at high temperatures (90°)
- Clean the Velcro hook side closure regularly of fluff and threads with a hard brush.
- Laundry bag is highly recommended to prevent loss of components during product washing and drying.

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