DE SERIELLE SCHNITTSTELLE (RS232)

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EN SERIAL INTERFACE (RS232)

In addition to the scope of functions described in the instructions for use, this scale is equipped with a serial interface (RS 232).

This interface allows data transfer between the scale and a dialysis system or a PC.

The interface cable included in the scope of supply for the scale is for connection to a dialysis system. The dialysis system interface must comply with the seca specification.

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The following accessory is required to connect the scale to a PC:

interface cable seca 451,
article number: 08-06-16-085

NOTE:

Model variants with an RS232 interface cannot be integrated in seca 360° wireless networks. This also applies to models whose standard variants belong to the seca 360° system, e.g. seca 704.

The above-mentioned accessory can be

obtained from seca customer service. The same department can also provide you with detailed information about operating the scale in conjunction with a dialysis system or a PC.

Visit us at www.seca.com or e-mail service@seca.com.

CAUTION!

Patient hazard, damage to device

Additional devices which are connected to medical electrical devices must provide evidence of compliance with the relevant IEC or ISO standards (e.g. IEC 60950 for data-processing devices). Furthermore, all configurations must comply with the require-

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ments of standards for medical systems (see IEC 60601-1-1 or Section 16 of the 3rd edition of IEC 60601-1 respectively).

Anyone connecting additional devices to medical electrical devices is considered a system configurator and is therefore responsible for ensuring that the system complies with the requirements of standards for systems.

Your attention is drawn to the fact that local laws take precedence over the above-mentioned requirements of standards. In the event of any queries, please contact your local specialist dealer or Technical Service.

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Fax · +49 (0)40 20 00 00 50

E-mail · info@seca.com

All contact details available at www.seca.com

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