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### EN

#### Intended Use:

The Urinal allows urination while lying in bed if the use of the toilet or any other aid such as toilet chair etc. is not possible.

#### Indication:

Strict bed rest, fractures (eg. of the pelvis), severe movement restrictions, cachexia

#### Contraindication:

physical insufficient or mental skills to use the aid (even despite assistance from a carer)

#### Description/ Versions

The Urinals are made of transparent or translucent plastic and are flattened on one side. This allows the product to be positioned in bed in front of the patient's genital area. The lid is only used for odour binding, it does not offer leak protection! The urine bottles are provided with a scale, which gives an overview of the excreted urine quantity (please use measuring containers intended for exact quantity). Urine bottles are available for men and women, they differ in shape of the bottle and opening. In addition, a plug-in Female Attachment for Male Urinals attachment for men's urine bottles is available, which allows use as a women's urine bottle. The product may only come into contact with intact, uninjured skin and must not be used across patients (exception: autoclavable models). Any use other than that mentioned above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user. Any serious incidents arising in connection with this product shall be reported to the manufacturer or the competent authority without delay.

#### Before use / maintenance:

First, read the present instruction manual in its entirety and keep it until the end of the product life. Please check before first use and any use whether the product is undamaged and make sure that the product has no sharp edges. In case of signs of damage or wear, the Urinal must not be used (any longer), please contact your dealer in this case.

#### Cleaning / disinfection:

Do not use sharp or abrasive utensils or detergents for cleaning. After each use, clean the urine bottle with a suitable bottle brush, warm water and soapy liquor or a commercially available, mild household cleaner. If necessary, the product can be disinfected with a commercially available, chlorine-free disinfectant in the wiping process. **Important: When using disinfectants, the manufacturer's application requirements must be observed.** After disinfection, rinse the urine bottle with plenty of clear water to remove any disinfectant residues. Then completely (let) dry.

Treatment for cross-patient use in nursing homes, hospitals, etc. (applies to autoclavable models 47 220 / 230): The urine bottle may only come into contact with intact, uninjured skin. According to the "Guideline of DGKH, DGSV, AK: Validation and routine monitoring of thermal cleaning and disinfection processes for medical devices ..." requires the application of an A0 value of 600

#### ⚠ Safety instructions:

- Before first use, clean the urine bottle and check for any damage.
- **The cover of the urine bottle only combats the odour and does not offer any leakage protection!**
- The caregiver should wear protective gloves.
- **After disinfection, the urine bottle must be rinsed with plenty of clear water in order to remove all disinfectant residues.**

Service Life: appr. 3 years, depending on frequency of use

#### Warranty:

According to the statutory warranty claims. Excluded from the guarantee are defects resulting out of natural wear and tear, intent, negligence or improper handling or use and defects which affect the value or usability of the tool only insignificantly. We reserve the right of modifications serving the technical improvement.

Distributed by

**puras**

031 996 85 85  
puras.ch