DE/EN Technische Daten / Technical specifications:

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EN User Manual

Intended Use:

The Female Urinal is a medical device designed for collecting urine and can allow women to urinate while sitting, lying or standing.

Indication:

Strict bed rest / bedridden, general states of weakness / cachexia, which do not allow to reach the toilet.

Contraindication:

Fractures that exclude the use, no sufficient physical or mental abilities to use the aid.

Description:

The Female Urinal is shaped like an oval shell with edge. The large opening and the integrated, hollow handle allow a comfortable application of the aid. The emptying of urine from the urine boat is carried out via the tubular handle, which is equipped with a removable closure. The product has a capacity of approx. 1 liter and is provided with a scale that gives an approximate overview of the amount of urine excreted (for exact quantity determination, please use measuring containers provided for this purpose).

Versions:

There are different aids for women to collect urine. The selection of the appropriate product should be carried out together with the specialized staff, since the use of the Female Urinal requires a certain skill and coordination. Testing and instruction in use must only be carried out by authorized, trained specialist personnel. The model is designed for domestic use, must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. For resulting damages there is no liability by Sundo GmbH, the risk is borne exclusively by the user. In the event of serious incidents occurring in connection with this product, please inform the manufacturer or the competent authority.

Before first use:

First of all, read the present instruction for use in full and keep it with the packaging until the end of the product's life. Please check before the first and each use by visual inspection whether the product is undamaged. In case of signs of damage or wear, the Female Urinal must not be used (any longer).

Use:

Take a comfortable sitting position or lying position in the supine position, if required use a bed protection or liquid-impermeable base to protect mattress or (wheel) chair. To collect the urine the Female Urinal is placed with the handle forward between the legs or under the anterior pelvis, depending on the position. After urination, the Female Urinal can be carefully pulled out again by pulling the handle. To empty, remove the plug and pour out the urine. The handling of the aid requires a certain amount of practice and skill.

Cleaning / Disinfection:

Do not use sharp or abrasive utensils or detergents for cleaning. After each use, clean Female Urinal and plug with a suitable bottle brush, warm water and soapy liquor or a commercially available, mild household cleaner. If necessary, the product can be disinfected with a commercially available, chlorine-free disinfectant. Please make sure that the Female Urinal and the plug (removed) are completely immersed in the disinfectant solution. **Important: When using disinfectants, the application regulations of the manufacturers must be observed.** After disinfection, rinse the urinal with plenty of clear water to remove any disinfectant residues. Then completely (let) dry.

⚠ Safety instructions:

- ▶ Caution: risk of suffocation and injury! Product and packaging (polybag) are not toys. Please do not leave children unattended with it!
- ▶ Please clean the Female Urinal completely before using it for the first time.
- ▶ Familiarize yourself with the use.
- ▶ Disposable gloves must be worn when assisted by a caregiver.
- ▶ After disinfection, rinse the product completely with plenty of clear water in order to be able to remove disinfectant residues completely.

Warranty:

According to the legal warranty claims. From the warranty excluded are damage by inappropriate use, normal wear or use as well as defects, that affect the value or usability of the device only insignificantly. Technical changes, alterations and extensions and improper maintenance. Work lead to expiring the warranty and the product liability. We reserve ourselves changes, which serve the technical improvement. **Note:** Repairs must only be carried out by trained personnel and using original spare parts.

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