## EN mylife Aveo™ Glucose Control Solutions

**IMPORTANT**

- Please read this information and the user guide of your mylife Aveo™ Blood Glucose Monitoring System (hereinafter referred to as BGMS) before using the mylife Aveo™ Glucose Control Solutions (control solution).
- Use control solution only with mylife Aveo™ Blood Glucose Meter (meters) and their corresponding mylife Aveo™ test strips (test strips).

**INTENDED PURPOSE**

The mylife Aveo™ Glucose Control Solutions are assayed quality control materials for use with the mylife Aveo™ Blood Glucose Monitoring System to check that the meter and test strips are working together properly and that the test is performing correctly.

**SUMMARY AND EXPLANATION**

The control solution contains a specific amount of glucose that is used to test meters and test strips. The control solution is for self-testing outside the body (*in vitro* diagnostic use). It may be used to check the performance of your meter and test strip or your testing technique. The test results you obtain should fall within the control solution ranges printed on the vial of test strips that you are using.

**CAUTION**

The control solution ranges printed on the test strip vial are for the two types of mylife Aveo™ Glucose Control Solutions only. These ranges are used to assess the meter and test strip performance. These are not recommended target ranges for your blood glucose level.

**USE CONTROL SOLUTION WHEN...**

- you want to practice the test procedure, without using blood,
- you use your meter for the first time,
- you begin using a new vial of test strips,
- you suspect that your meter or test strips are not working properly,
- you think your test results are inaccurate or do not reflect how you feel,
- you dropped your meter.

**STORAGE AND HANDLING**

Store the control solution at temperatures between 8–30 °C, tightly capped in the bottle. It must be kept at room temperature (20–25 °C) before testing to obtain correct results. Do not refrigerate or freeze the control solution. Do not use beyond the expiration date shown on the bottle. Use within 3 months (90 days) of opening.

**WARNINGS AND PRECAUTIONS**

- The control solution is designed for *in vitro* diagnostic use only.
- The control solution is intended to be used only for self-diagnostic testing with the dedicated meters and their corresponding test strips.
- Do not swallow or ingest control solution.
- Check the expiration date on the bottle and do not use it if expired.
- Discard expired control solution safely in appropriate containers according to the regulations applicable in your country.
- If the test results keep falling outside the specified range continually, do not use your BGMS to test blood glucose.
- Make sure your meter, test strips, and control solution are at room temperature before testing. Control solution tests must be done at room temperature (20–25 °C).
- Shake the bottle well before use, discard the first drop of control solution and wipe the tip with a clean tissue before applying the control solution to the test strip.
- The control solution contains a dye that may stain clothes.
- In the event of any serious incident with the mylife Aveo™ Glucose Control Solution, please report to manufacturer and the competent authority of the Member State.

**TEST PROCEDURE**

For detailed instructions on how to perform a control solution test, please refer to the user guide of your BGMS.

**INTERPRETATION OF OUT OF TARGET RANGE RESULTS**

At room temperature, the control solution test results should fall within the control solution range printed on the test strip vial about 95 % of the time. If your control solution test result doesn't fall within the expected range, then repeat the test.

Out-of-range results may be due to one or more of the following factors:

- Error in the way the test was performed
- Failure to shake the control solution bottle vigorously or to wipe the vial tip clean
- Control solution expired or contaminated
- Test strip wrong, damaged or expired
- Control solution, test strip or meter were exposed to high or low temperatures
- Meter is not functioning properly

For further information, please refer to the user guide of your BGMS.

**IMPORTANT**

You continue to get control solution test results that fall outside the range printed on the test strip vial, the BGMS may not be working properly and should therefore not be used for blood glucose testing. If you are unable to resolve the problem, contact your local distributor's customer service.

**CHEMICAL COMPOSITION**

Each mylife Aveo Glucose Control Solution low contains the following reagents:

- 2-(N-Morpholino)ethansulfonic acid: 10,7 g/L
- D-(+)-Glucose: 0,28 g/L

Each mylife Aveo Glucose Control Solution normal contains the following reagents:

- D-(+)-Glucose: 0,62 g/L
- 2-(N-Morpholino)ethansulfonic acid: 10,7 g/L

Each mylife Aveo Glucose Control Solution high contains the following reagents:

- D-(+)-Glucose: 1,4 g/L
- 2-(N-Morpholino)ethansulfonic acid: 10,7 g/L

**METROLOGICAL TRACEABILITY**

The system has been evaluated using the YSI glucose analyzer as the reference method and traceable to the NIST Standard Reference Material (SRM) 917D. Using the traceability chain, the results obtained with test strips for control solutions can also be traced back to the NIST standard.

**ANNOTATIONS**

- No part of this document may be reproduced in any form or by any means without the prior written consent of i-SENS.
- The information in this insert is correct at the time of printing. However, i-SENS reserves the right to make any necessary changes at any time without notice as we are continuously improving our products.
- The Summary of Safety and Performance (SSP) is accessible to the public via the European Database on Medical Devices (EDAMED). Search for the currently available SSPs using the specified search criteria detailed on the following website: *https://ec.europa.eu/tools/edamed/#/screen/search-device*

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