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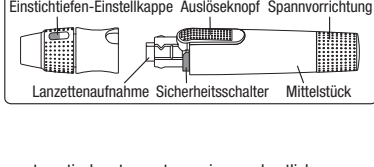
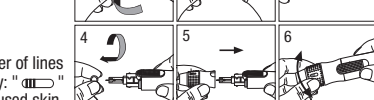
Rev. Date: 2023-05-05 (Version 01)

## mylife Softlance - Instructions for the lancing device EN

The Lancing Device is a reusable lancet device intended for acquiring capillary blood samples for the purpose of blood glucose testing.

The Lancing Device is recommended for use by a single user. If shared between multiple users, the Lancing Device must be cleaned and disinfected between each use. A new disposable lancet carrier must be used for each sampling. A depth adjustable cap allows the penetration depth to be tailored to the requirements of each user.

### Special features

- Special Safety Design: Removing the cap automatically releases the trigger to prevent accidental pricking if the release button is pressed.
- Lancet Ejection Function: Hold down the release button and pull the plunger at the same time to eject the used lancet.
- Less Painful Pricking: Vibration-free lancet guidance technology makes pricking almost painless.
- Appropriate Spring Force: Pulling back the plunger loads the spring with sufficient force to prick your skin easily without excessive pain.
- Clear indication of depth penetration: allow user to adjust for proper sample volume easily.E3

### Cautions

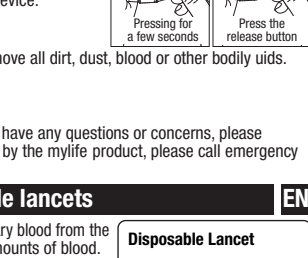
- Be careful not to be pricked by the lancet, when removing the depth adjustable cap.
- The Lancing Device is intended for use by a single user. If used by another user, the Lancing Device must rst be cleaned and disinfected.
- Do not disassemble the lancet device other than as specified in the Instructions for Use.
- Storage temperature: -10 - 60°C (14 - 140°F).

### Warning

The lancet device contains some small parts which may present a choking hazard for children, please do not place the device in a place where children can easily reach it.


### Instructions for use (Wash and dry your hands.)

1. Hold the depth adjustable cap in one hand and hold the hub in the other hand. Bend the cap towards the down side, until a gap appears between the cap and hub.
2. Pull the cap and hub off in opposite directions, remove the cap.
3. Insert a new disposable lancet firmly into the lancet carrier.
4. Twist off and set aside the protective cap of the disposable lancet.
5. Reconnect the depth adjustable cap to the hub.
6. Rotate the clear top part of the cap to adjust the depth. Check the number of lines visible in the View Window. More lines correspond to a greater depth. Try: "III" for soft or thin skin; "IIII" for ordinary skin; "IIIIII" for thick or calloused skin.
7. Pull back the plunger until you hear a click. The device is now primed. Let go of the plunger. It will return to its original position.
8. Place the end of the lancing device against your fingertip and press the release button to release. You can use this blood sample to perform a blood glucose test.

### Lancet Disposal

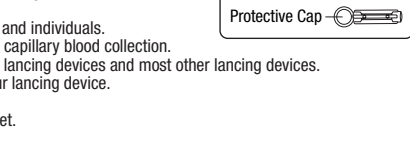1. Separate the cap from the hub. Press the lancet needle into the side of its original lancet cap.
2. (1) Hold down the release button and (2) pull back the plunger to eject the lancet.
3. Dispose the lancet into a suitable biohazard container.
4. Replace the depth adjustable cap after nishing the test.

### Alternative site testing-palm or forearm blood sampling

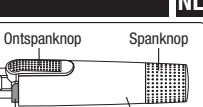
- Change the depth adjustable cap to the clear cap of your Lancing Device.
- To increase blood ow, massage the target sample area for a few seconds.
- Press and hold the lancet device against the target area and press the release button.
- Continue holding the lancet device against the target area.
- Gradually apply more pressure until you can see the droplet is big enough (refer to Test Strip Insert).

### Note: For blood glucose testing procedure, please refer to User's Manual or Test Strip Insert.

### Cleaning and Disinfecting (Frequency: at least once a week)

- Wipes using Isopropanol as the active ingredient are safe for use with the lancet device. (Tested brand: Caviwipes, manufacturer: Metrex)

### Procedure

1. Wipe down the entire surface of the lancet device using disinfecting wipes to remove all dirt, dust, blood or other bodily uids.
2. Take a new disinfecting wipe and wipe the meter thoroughly again.
3. Allow the surface to remain wet for 2 minutes.

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