BRAUN

ThermoScan®

Ear thermometer

Distributed by

puras


031 996 85 85
puras.ch

IRT 6020
IRT 6520

GB English	2
DE Deutsch	9
ES Español	16
FR Français	25
GR Ελληνικά	33

IT Italiano	41
NL Nederlands	49
PT Português	57
SA اللغة العربية	71
TR Türkçe	72

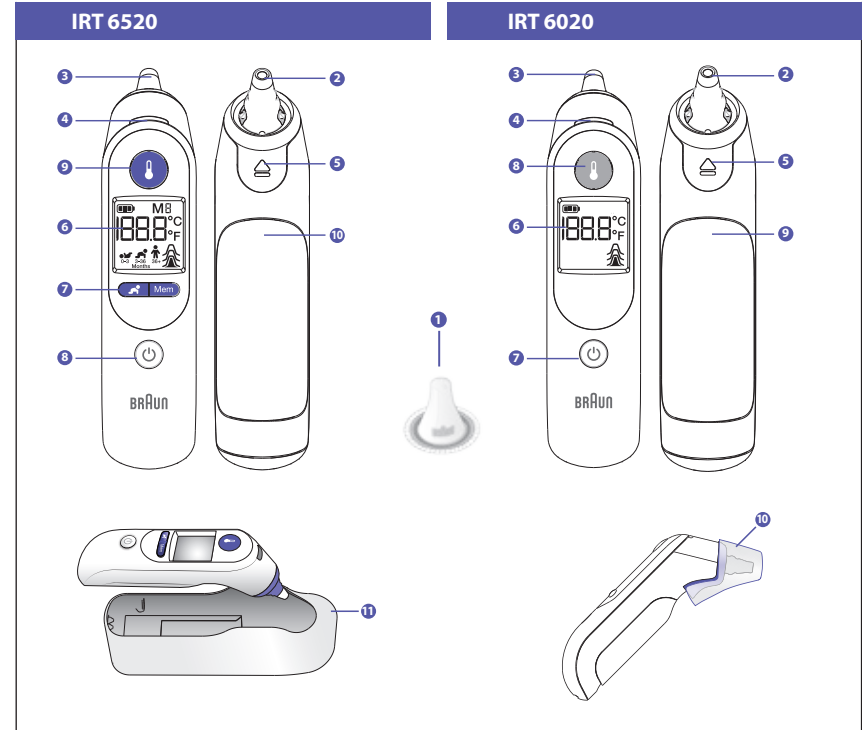This product is manufactured under a license to the 'Braun' trademark. 'Braun' is a registered trademark of Braun GmbH, Kronberg, Germany.

ThermoScan®, ExacTemp® and Age Precision® are registered trademarks of Helen of Troy Limited and/or its affiliates.

Kaz Europe Sàrl
 A Helen of Troy Company
 Place Chauderon 18
 CH-1003 Lausanne
 Switzerland

Printed in USA

© 2018, All rights reserved.

www.hot-europe.com

IRT6020/6520 MNL-WE
 P/N: A002877R0
 21AUG18

Patents: www.kaz.com/patents/braun

Made in Mexico

Product description

IRT 6520

- 1 Disposable Hygiene cap
(a box of 20 is supplied + 1 already installed)
- 2 Probe tip
- 3 Probe
- 4 ExacTemp® light
- 5 Hygiene cap ejector
- 6 Display
- 7 Age Precision® and Memory buttons
- 8 Power button
- 9 Start button
- 10 Battery door (2 X 1.5V type AA (LR6) batteries)
- 11 Protective case

IRT 6020

- 1 Disposable Hygiene cap
(a box of 20 is supplied + 1 already installed)
- 2 Probe tip
- 3 Probe
- 4 ExacTemp® light
- 5 Hygiene cap ejector
- 6 Display
- 7 Power button
- 8 Start button
- 9 Battery door (2 X 1.5V type AA (LR6) batteries)
- 10 Protective cap

The Braun ThermoScan® Ear thermometer is indicated for intermittent measurement and monitoring of human body temperature for people of all ages. It is intended for household use only.

Use of IRT 6520 along with Age Precision® feature is not intended as a substitute for consultation with your physician.

Warnings and Precautions

- Please consult your physician if you see symptoms such as unexplained irritability, vomiting, diarrhea, dehydration, changes in appetite or activity, seizure, muscle pain, shivering, stiff neck, pain when urinating, etc. in spite of any colour of the background light or absence of fever.
- Please consult your physician if the thermometer shows elevated temperature (yellow or red background lights for IRT 6520).

The operating ambient temperature range for this thermometer is 10 – 40 °C (50 – 104 °F). Do not expose the thermometer to temperature extremes (below -25 °C / -13 °F or over 55 °C / 131 °F) or excessive humidity (>95% RH). This thermometer must only be used with genuine Braun ThermoScan disposable Hygiene caps (LF 40).

To avoid inaccurate measurements always use this thermometer with a new, clean Hygiene cap attached.

If the thermometer is accidentally used without a Hygiene cap attached, clean the lens (see «Care and cleaning» section). Keep Hygiene caps out of reach of children.

This thermometer is intended for household use only. This product is not intended to diagnose any disease, but is a useful screening tool for temperature. Use of this thermometer is not intended as a substitute for consultation with your physician.

The Age Precision feature is not intended for pre-term babies or small-for-gestational age babies.

The Age Precision feature is not intended to interpret hypothermic temperatures. Do not allow children under 12 to take their temperatures unattended.

Do not modify this equipment without authorization of the manufacturer.

Parents/guardians should call the pediatrician upon noticing any unusual sign(s) or symptom(s). For example, a child who exhibits irritability, vomiting, diarrhea, dehydration, seizure, changes in appetite or activity, even in the absence of fever, or who exhibits a low temperature, may still need to receive medical attention.

Children who are on antibiotics, analgesics, or antipyretics should not be assessed solely on temperature readings to determine the severity of their illness.

Temperature elevation as indicated by Age Precision may signal a serious illness, especially in adults who are old, frail, have a weakened immune system, or neonates and infants. Please seek professional advice immediately when there is a temperature elevation and if you are taking temperature on:

- neonates and infants under 3 months (consult your physician immediately if the temperature exceeds 37.4 °C or 99.4 °F)

- patients over 60 years of age
- patients having diabetes mellitus or a weakened immune system (e.g. HIV positive, cancer chemotherapy, chronic steroid treatment, splenectomy)
- patients who are bedridden (e.g. nursing home patient, stroke, chronic illness, recovering from surgery)
- a transplant patient (e.g. liver, heart, lung, kidney)

Fever may be blunted or even absent in elderly patients.

This thermometer contains small parts that can be swallowed or produce a choking hazard to children. Always keep the thermometer out of children's reach.

Age Precision® colour-coded display

IRT 6520 Only

Clinical research shows the definition of fever changes as newborns grow into children and children grow into adults¹.

Braun's Age Precision colour-coded display takes the guesswork out of interpreting temperature for the entire family.


Simply use the Age Precision button to select the appropriate age setting, take a temperature and the colour-coded display will glow green, yellow or red to help you understand the temperature reading.

¹ Herzog L, Phillips S. Addressing Concerns About Fever. *Clinical Pediatrics*. 2011;50(5): 383-390.

How to use your Braun ThermoScan®

1. IRT 6520

Remove the thermometer from the protective case.

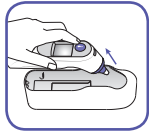IRT 6020

Remove protective cap.

2. Push the Power button .


During an internal self-check, the display shows all segments. Then the last temperature taken will be displayed for 5 seconds.

3. The Hygiene cap indicator will blink to signal one is needed.

To achieve accurate measurements, make sure a new, clean Hygiene cap is in place before each measurement.

Attach a new Hygiene cap by pushing the thermometer probe straight into the Hygiene cap inside the box and then pulling out.



Note: The Braun ThermoScan will not work unless a Hygiene cap is attached.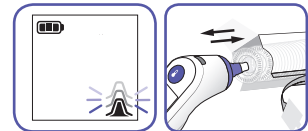

4. IRT 6520


Select age with Age Precision button  as shown below. Press button to toggle through each age group.

NOTE: You must select an age in order to take a measurement.

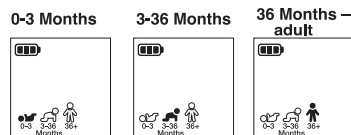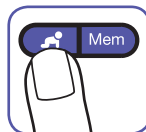
The thermometer is ready to take a temperature when the display looks like the images below.

IRT 6520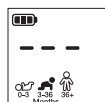

IRT 6020

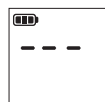5. Fit the probe snugly into the ear canal and direct towards opposite temple. ExacTemp® will confirm stability and indicate consistent probe position with a light and a beep. Then push and release the Start button .


6. ExacTemp® light will pulse while temperature measurement is in progress. The light will remain solid for 3 seconds to indicate that a successful temperature reading has been achieved.

NOTE: If the probe has been properly inserted into the ear canal during the measurement, a long beep will sound to signal the completed measurement.


If the probe has not been constantly placed in a stable position in the ear canal, a sequence of short beeps will sound, the ExacTemp light will go out and the display will show an error message (POS = position error).

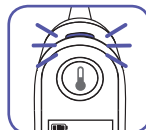See «Errors and troubleshooting» section for more information.

7. The confirmation beep indicates that an accurate temperature measurement has been taken. The result is shown on the display. Temperature reading displayed is an equivalent of oral temperature.

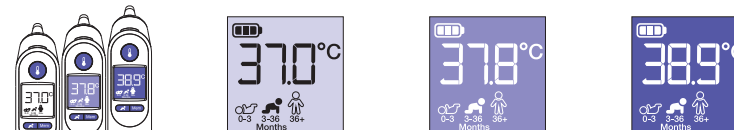
IRT 6520

Age range	Green Normal temperature	Yellow Fever	Red High fever
 0-3 Months	≥35.8 – ≤37.4 °C (≥96.4 – ≤99.4 °F)	>37.6 – ≤38.5 °C (>99.6 – ≤101.3 °F)	>37.4 °C (>99.4 °F)
 3-36 Months	≥35.4 – ≤37.6 °C (≥95.7 – ≤99.6 °F)	>37.6 – ≤38.5 °C (>99.6 – ≤101.3 °F)	>38.5 °C (>101.3 °F)
 36 Months - adult	≥35.4 – ≤37.7 °C (≥95.7 – ≤99.9 °F)	>37.7 – ≤39.4 °C (>99.9 – ≤103.0 °F)	>39.4 °C (>103.0 °F)

IRT 6020

Temperature displays.
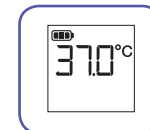


8. For the next measurement, press Eject button  to remove and discard used Hygiene cap, and put on a new, clean Hygiene cap.

NOTE: Defaults to last age setting used if you do not change.

The Braun ThermoScan Ear thermometer turns off automatically after 60 seconds of inactivity. The thermometer can also be turned off by pressing the Power button .

The display will briefly flash OFF and it will go blank.

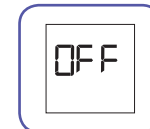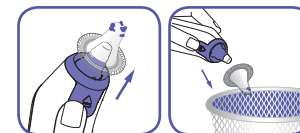Temperature-taking hints

Replace disposable Hygiene cap with every use to maintain accuracy, hygiene and to prevent cross-contamination. Earwax and dirt on the tip of the Hygiene cap, even in a microscopic layer, can create an obstacle between the sensor and the infrared heat emitted by the ear and impact the accuracy of the measurement.

External factors like poor probe replacement, used Hygiene cap or dirty Hygiene cap may influence ear temperature measurements.

In clinical studies, precision and accuracy of ear thermometer measurement has been shown to be influenced by variability in user technique. **Correct probe placement is essential for accurate measurements.**

Temperature in right and left ear differs, always compare measurements from the same ear. The natural physiological difference may reach 0.2 °C. In the absence of confounding factors, in high-risk population (like infants or immunodepressed people) use the highest of the two to interpret the patient's temperature.



In the following cases, wait 20 minutes prior to taking a temperature: extreme changes in room temperature, hearing aid, lying on a pillow. Use the untreated ear if ear drops or other medications have been placed in the ear canal.

Memory mode


IRT 6520

- This model stores the last 9 temperature measurements. To display the stored measurement, the thermometer must be turned on.
- Press the **Mem** button.
The display shows the temperature, and when releasing the **Mem** button, the stored temperature for that memory number is displayed, together with "M".
- Each time the **Mem** button is pushed, the stored reading and an M is displayed to indicate each temperature measurement (eg. M2).
Memory mode is automatically exited by not pressing the memory button for 5 seconds.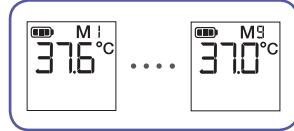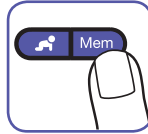

IRT 6020

- The last temperature taken is stored in its memory and will be automatically displayed for 5 seconds when it is turned on again.

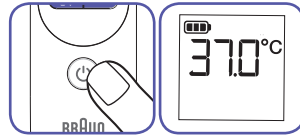Night light feature

IRT 6520 Only

The thermometer includes a convenient night light to illuminate the display in a dim environment.


The light will turn on when you press any button. It will stay on until the thermometer is inactive for up to 15 seconds even after temperature is taken.

Changing the temperature scale

Your Braun ThermoScan is shipped with the Celsius (°C) temperature scale activated. If you wish to switch to Fahrenheit (°F) and/or back from Fahrenheit to Celsius, proceed as follows:

- Make sure the thermometer is turned off.
- Press and hold down the Power button . After about 3 seconds the display will show this sequence: °C / SET / °F / SET.
- Release the Power  button when the desired temperature scale is shown. There will be a short beep to confirm the new setting, then the thermometer is turned off automatically.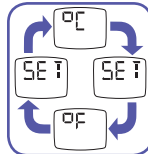

Care and cleaning

 The probe tip is the most delicate part of the thermometer. It must be clean and intact to ensure accurate readings. If the thermometer is ever accidentally used without a Hygiene cap, clean the probe tip as follows:

Very gently wipe the surface with a cotton swab or soft cloth moistened with alcohol. After the alcohol has completely dried out, you can put a new Hygiene cap on and take a temperature measurement.

If the probe tip is damaged, please contact Consumer Relations.

Use a soft, dry cloth to clean the thermometer display and exterior. Do not use abrasive cleaners. Never submerge this thermometer in water or any other liquid. Store thermometer and Hygiene caps in a dry location free from dust and contamination and away from direct sunlight.

Additional Hygiene caps (LF 40) are available at most stores carrying Braun ThermoScan.



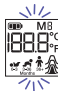
Replacing the batteries

The thermometer is supplied with two 1.5 V type AA (LR 06) batteries.

- Insert new batteries when the battery symbol appears on the display.
- Open the battery compartment. Remove the batteries and replace with new batteries, making sure the poles are in the right direction.
- Snap battery door into place.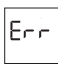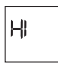

Errors and troubleshooting

Error Message	Situation	Solution
	No Hygiene cap is attached.	Attach new, clean Hygiene cap.
	The thermometer probe was not positioned securely in the ear. An accurate measurement was not possible. POS = position error	Take care that the positioning of the probe is correct and remains stable. Replace Hygiene cap and reposition. Press Start button to begin a new measurement.
	Ambient temperature is not within the allowed operating range (10 – 40 °C or 50 – 104 °F).	Allow the thermometer to remain for 30 minutes in a room where the temperature is between 10 and 40 °C or 50 and 104 °F.
	Temperature taken is not within typical human temperature range (34 – 42.2 °C or 93.2 – 108 °F).	Make sure the probe tip and lens are clean and a new, clean Hygiene cap is attached. Make sure the thermometer is properly inserted. Then, take a new temperature.
	HI = too high LO = too low	
	System error – self-check display flashes continuously and will not be followed by the ready beep and the ready symbol. If error persists, If error still persists,	Wait 1 minute until the thermometer turns off automatically, then turn on again. ... reset the thermometer by removing the batteries and putting them back in. ... please contact the Service Centre.

Error Message	Situation	Solution
	Battery is low, but thermometer will still operate correctly.	Insert new batteries.
	Battery is too low to take correct temperature measurement.	Insert new batteries.

Product specifications

Displayed temperature range:	34 – 42.2 °C	(93.2 – 108 °F)
Operating ambient temperature range:	10 – 40 °C	(50 – 104 °F)
Storage temperature range:	-25 – 55 °C	(-13 – 131 °F)
Operating and storage relative humidity:	10-95% RH (non condensing)	
Display resolution:	0.1 °C or °F	
Accuracy for displayed temperature range	Maximum Laboratory Error	
35 – 42 °C (95 – 107.6 °F):	±0.2 °C	(±0.4 °F)
Outside this range:	±0.3 °C	(±0.5 °F)
Clinical repeatability:	±0.14 °C	(±0.26 °F)
Battery life:	2 years / 350 measurements	
Service life:	5 years	

This thermometer is specified to operate at 1 atmospheric pressure or at altitudes with an atmospheric pressure up to 1 atmospheric pressure (700 - 1060 hPa).

Equipment with type BF applied parts

See instruction for use

Operating temperature

Storage temperature

Keep it dry

Subject to change without notice.

This product conforms to the provisions of the EC directive 93/42/EEC. This product carries the CE mark and is manufactured in conformity with the RoHS Directive 2011/65/EU.

MEDICAL ELECTRICAL EQUIPMENT needs special precautions regarding EMC. For detailed description of EMC requirements please contact your authorized local Service Centre (See guarantee card).

Portable and mobile RF communications equipment can affect MEDICAL ELECTRICAL EQUIPMENT.

 This product contains batteries and recyclable electronic waste. To protect the environment, do not dispose of it in the household waste, but take it to appropriate local collection points.

Calibration

This product has been calibrated at the time of manufacture. If used according to the use instructions, periodic calibration is not required. If at any time, you question the measuring accuracy, please contact our service representative (see last page for contact information) or visit www.hot-europe.com/support.

Manufacturing date is given by the LOT number located in the battery compartment. The first three (3) digits after LOT represent the Julian date that the product was manufactured and the next two (2) digits represent the last two numbers of the calendar year the product was manufactured. The last identifiers are the letters that represent the manufacturer.

An example: LOT 11614k, this product has been manufactured on the 116th day of the year 2014.

Guarantee

EMC Tables and Consumer Card are available on our website at www.hot-europe.com/support.

For support requests, please visit www.hot-europe.com/support or find service contact information on the last page of this manual.

UK only: This guarantee in no way affects your rights under statutory law.

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