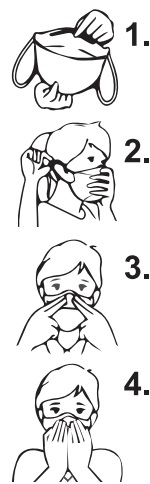
FFP2 maschera

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
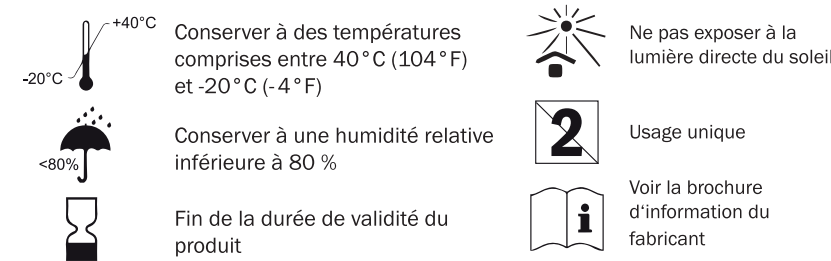
FFP2 mask

Instructions for use

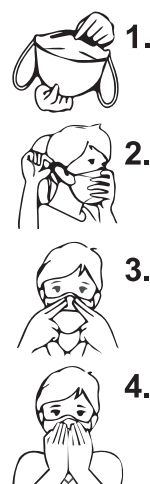
Before opening and touching the mask, wash your hands thoroughly with soap and water or alcohol-based hand sanitizer (>60% alcohol content).
Open the mask, one hand on each side.
Fit the mask over your mouth and nose and pull the straps over your head.
Press the nose bridge clip to fit your nose.
Check for a good adhesion between the mask and your face. Cover the mask with both hands, inhale and exhale strongly to check for possible air leakage from the mask edge. Adjust as necessary.

Precautions

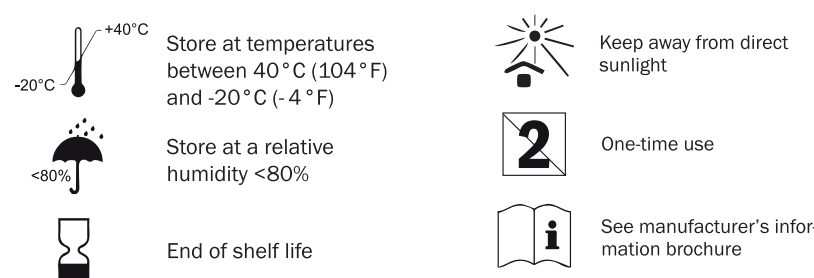This mask is for one-time use (max. 8 hours). Do not reuse mask. This product is not suitable for protection against harmful gases, steam and oily aerosols.
The product does not generate oxygen and is not suitable for use in explosive atmospheres or anoxic environments, underwater operations, during evacuation or fire fighting environments.
Do not use mask if you do not feel a good adhesion effect between the face and the mask edge during use.
If found during use that the mask is damaged, replace with a new one. Leakage requirements are unlikely to be met if facial hair is under the sealing line on the face.

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Furber AG, Hintermättlistrasse 3, CH-5506 Mägenwil, +41 (0)62 889 80 80