EN User Manual

Intended use

The bottle holder can be hung on the patient's bedside for convenient storage of various bottles. Bottles are placed in a tilt-proof manner, and independent handling by the patient can also be made possible. If this is not desired, the bottle holder can be installed on the bedside outside the patient's reach.

The bottle holder must only come into contact with intact, uninjured skin and must not be used across patients. Any use other than that mentioned above is not intended. Sundo GmbH is not liable for any resulting damages, the risk is borne exclusively by the user.

Types:

There are different models / sizes for different bottles (see picture), please ask your specialist dealer for advice.

Proir to use / Maintenance:

First, read this manual in its entirety and keep it until the end of the product's life. Please check whether the product is undamaged before first and each use by visual inspection. Moving parts are also tested for smooth running functionality. If there are signs of damage or wear, the bottle holder must not /no longer be used. No further maintenance is required.

For use with urine bottles:

- Hang the bottle holder in the bed frame with the curved part facing down. If the patient can use the urine bottle independently, the holder should be placed within reach so that the bottle can be easily used.
- After use, the urine bottle can be placed back in the holder. **Item No. 47 300:** the additional lid of the holder attached to the chain can be used as a closure (**attention, to prevent odours, no leak protection!**)
- The caregiver can then remove the urine bottle and empty it in the toilet.
- After emptying, clean the urine bottle and disinfect it if necessary.

Cleaning and disinfection:

- The bottle holder should be cleaned or disinfected at least once a day, more often if necessary.
- Any impurities should be removed immediately in warm soapy water or with a damp cloth and suitable, mild cleaner to avoid drying.
- Disinfect the bottle holder with a commercially available, chlorine-free disinfectant before first and after each use. Important: please observe the manufacturer's application regulations.
- Dry the bottle holder well and hang it back on the bed frame for further use.

⚠ Safety

- The bottle holder must only come into contact with intact, uninjured skin!
- The bottle holder is only designed for domestic use and not for cross-patient use.
- Before first use, the bottle cage must be cleaned and checked for possible damage.
- The lid of the bottle cage is only used to prevent odours and does not offer leak protection!
- The caregiver should wear protective gloves.

Warranty

According to the legal warranty claims. Excluded from the warranty are damages caused by improper use, normal wear and tear and consumption as well as defects that have only a negligible influence on the value or usability of the product. Technical changes, conversions and extensions as well as improper maintenance work lead to the expiry of the warranty and product liability. We reserve the right to make changes that serve the purpose of technical improvement.

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